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Rhinose Rhinosectan®

PLEASE READ THIS LEAFLET CAREFULLY

What is Rhinose and what is it used for?

Rhinose is a medical device that forms a protective barrier over the nasal epithelial mucosa, thus the contact of allergens and pathogens with the mucosa is reduced or impeded, preventing or minimizing the development of nasal symptoms. Its protective action is principally due to the presence of xyloglucan a natural polymer showing the capability to protect the epithelia and reduce the hypersecretion of liquids such as mucus. Rhinose is specifically formulated to control and treat the symptoms of rhinitis and rhinosinusitis from different aetiologies, especially those of allergic origin. Rhinose relieves the symptoms of allergies caused by pollen, dust mites or pets acting as a barrier to the penetration of airborne allergens. Rhinose can also be taken as a preventive measure before coming into contact with an environment where airborne allergens are likely to be present (flowering periods, gardening activities, home dusting, et al.) as its mode of action provides protection from the very beginning, before symptoms appear.

How to use Rhinose

Always administer Rhinose according to the instructions for use. Consult your healthcare provider or pharmacist before taking it in case of doubt or if symptoms persist. Apply 1-2 puffs in each nostril 3 or 4 times a day. Regular use is recommended when first symptoms are likely to appear and during the course of rhinitis. The dosage may be increased according to severity of symptoms.

1. Gently blow your nose to clean your nostrils
2. Shake the bottle and remove the protecting cap
3. Hold the bottle with your index and middle finger on both sides and your thumb underneath the bottle. Bend your head slightly forward and close one nostril by gently pressing against the side of your nose with the finger.
4. Insert the applicator into your nostril and press down on the nozzle to puff once or twice
5. Insert the applicator into your other nostril and repeat the same steps described above.
6. Recap the bottle

The nozzle used in Rhinose provides a droplet size suitable to ensure a narrow jet thus allowing the adequate distribution of the solution in the whole nasal cavity, reaching even the upper part.

Possible side effects of Rhinose

Rhinose is usually well tolerated. Up to now, no regularly occurring or persisting side effects are known. Please inform your healthcare provider if you notice side effects that are associated with the use of Rhinose.

Precautions

Rhinose should not be used in case of hypersensitivity to xyloglucan or to any other ingredient of the product. If allergic reactions occur, the use of Rhinose must be discontinued immediately. To ensure safe hygiene, each bottle should only be used by one person. Do not use for more than 6 weeks after first opening. Do not use the device if damaged. Rhinose does not contain any pharmacological ingredient such as antihistamines or steroids. Should another pharmacological treatment be needed, Rhinose can be administered concomitantly with it.

What should be noted during pregnancy and while breast-feeding?

Even though no side effects are known, consult your healthcare provider or pharmacist for advice before using this medical device if you are pregnant or breastfeeding.

How to store Rhinose

The expiry date of Rhinose is printed on the box and on the bottle. Rhinose must not be used after the expiry date. Rhinose does not require any special storage conditions. Do not freeze. Rhinose should be kept out of the reach of children.

Composition:

Ingredients: Xyloglucan (0.15g/15ml), potassium sorbate, sodium EDTA, boric acid, sodium hydroxide, and saline solution.

Presentation:

Bottle containing 15 ml solution for nasal application.

Rev.00: 02.09.2019

 **Manufacturer:** DEVINTEC SAGL
Via Berna, 9, 6900 Lugano - Switzerland
Importer and Registration Holder:
Lapidot Medical Import and Marketing Ltd.,
8 Hashita St., Caesarea.
Telephone number: 1700-70-90-33

Approved by the Ministry of Health, Medical Device Department. Registration No.: 4350619

Rhinosectan®	DIMENSIONS (mm)	PRINTABLE COLOURS / PANTONES: 1	ELEMENT TO ADD TO THE PACKAGE:
PRESENTATION: Leaflet 15 ml spray	Length: 160	Black	 DEVINTEC SAGL Via Berna 9 6900 Lugano, Switzerland
COUNTRY: ISRAEL	Width: -		
CODE: C31E06H015IL	Height: 210		
	SUPPLIER INFO: Ferrer		 0373 Rev.XX: dd.mm.yyyy
	Supplier Product info :		
Revision date: 30.10.2019			